## USER ISTRUCTIONS REINFORCED SURGICAL GOWN STERILE EO

Single-use medical device  
(Sterile REF AH-DM 17-01)

Composition: Non-woven textile fabric of multi-layer laminate filter of 43g /m2 type SMMS, 100% PP (4 layers: one layer of Spunbond, 2 layers of Meltblown and one of Spunbond) color Blue, also reinforced at the sleeves and in the front middle made from PP+PE of 37g/m2 (polypropylene +polyethylene laminate layer) with very high resistance to liquid penetration, white color.  
Read this leaflet carefully and completely before you begin to use this medical device, because the leaflet includes important information for you.

- Keep this leaflet. You may have to read it again.
- If you have additional questions, contact the manufacturer.
- If you show potential adverse reactions not mentioned in this leaflet, please contact a physician or a pharmacist.
- Always use this medical device according to the indications in this leaflet or to the manufacturer's recommendations.
- For additional information/ recommendations, contact manufacturer.

What you will find in this leaflet:

1. What is the Reinforced Surgical Gown, and which are its uses?
2. Things you need to know before you use the Reinforced Surgical Gown
3. How to use the Reinforced Surgical Gown
4. Potential side effects
5. How to store the Reinforced Surgical Gown

1. What is Reinforced Surgical Gown, and which are its uses?

Surgical Kittel is a medical device for use by medical staff in outpatient care providers and operating rooms working in sterile areas and protects against contamination of infectious aggregations, ensuring a high degree of protection.

The reinforced surgical gown is a High Performance product for use in critical area. It is sterile, waterproof, airtight, easy to take on and comfort.

The surgical Gown is manufactured in accordance with the requirements of the European Directive no. 93/42/EEC Annex V, on the marketing of medical devices, and the standard EN 13795-1:2019.  
Sterile Surgical Gown is produced under internal conformity assessment procedure QMS, ISO -13485 and under surveillance CE notified sterile process (UDEM-CE nr 2292 CE Marking Certificate No:M.2021.106.14599)

Presentation: Reinforced Surgical Gown is available in S M L XL XXL sizes, single pack in a sterilization paper (polyethylene/paper)

Dimensions:	S	M	L	XL	XXL
Front Length, Snp length	123	123	133	143	143
Centre back Length	121	121	131	141	141
CF Reinforced width	46	46	49	49	52
CF Reinforced Length	60	60	80	80	90

### 2. Before use

Do not use the Reinforced Surgical Gown if you are allergic to polypropylene !

### 3. How to use the Reinforced Surgical Gown

Read below safety description before use

- Clean and disinfect your hands!
- Open the package.
- Carefully unpack the gown.
- Put the Surgical gown on by tucking your hands into the sleeves..
- Tie the strings and ensure that the gown is closed tightly.
- After use, untie the strings and undress the gown without touching the clothes underneath.
- The used disposable Surgical gowns should be disposed as hazardous medical waste in accordance with Executive Order No. 1226/2012 on the handling of hazardous medical waste.

### Use by children and adolescents

The Surgical Gown is not meant to be used by children or adolescents.

### 4. Potential side effects

No side effects have been identified until now.

Material selected with care for minimum risk of allergy and hypersensitivity, but a guarantee cannot be given for everyone.

No adverse side effects have been reported in children or adolescents.

No side effects are known in relation to the use of the Surgical Gown together with other medicines or disinfectants applied on the skin, nor with other protective medical devices.

If you show any kind of adverse reaction, please contact your physician or pharmacist! Such reactions will include any adverse reactions not mentioned in this leaflet.

### Contraindications

No contraindications are known.

If an abnormal skin reaction appears, stop using the product immediately

### Warnings and precautions

- Do not use this product for other purposes than those mentioned in the leaflet.
- Before you put on the surgical gown, wash your hands and disinfect with a skin disinfectant, in order to prevent contamination when you dress.
- Use the product immediately after you open it.
- Do not use the product, if the gown is damaged or shows signs of use.
- Do not use after the expiry date shown on the package. The expiry date concerns the last day of the indicated month.
- Protect from sunlight, humidity, dust and heat / flame sources.
- Store at temperature between 5 and 25 °C and a humidity of maximum 65%.
- Keep away from children.
- Recommended exclusively for use in sterile environment
- The product is Sterile, meant to be used only once. Do not re sterilize.
- If abnormal skin reactions appear, stop using the product immediately and contact your physician or pharmacist.

### Pregnancy and breastfeeding

The Reinforced Surgical Gown can be used by pregnant women or women who are breastfeeding. If you believe you may be or intend to become pregnant, contact your physician for recommendations before you use this medical device.

### Vehicle driving

The Surgical Gown does not have an impact on the driving of motor vehicles.

### 5. Storage

- Keep this device away from children.
- Protect from humidity, heat, dust and extended exposure to sunlight.

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EMS MEDICAL A/S, DENMARK
EMS medical A/S or our local representative agent.